



Important Notes

- Before installing this software on your computer, verify that the computer meets minimum specifications and that you have administrative rights to install new software.
 - See the *Welch Allyn® Diagnostic Cardiology Suite ECG Specifications* or the *Welch Allyn® Diagnostic Cardiology Suite Modality Manager User Manual, Appendix A*, for minimum specification requirements.
- Install the Welch Allyn® Diagnostic Cardiology Suite Modality Manager software first before installing the Modality/Modalities software.
 - The USB flash drive includes all purchased software programs.
- Schedule the software installation allowing sufficient time to complete the installation without interrupting patient exams and to restart and configure your computer when the installation completes.
- If prompted, install any drivers related to the acquisition devices supported by the system.
 - Select **Install this driver software anyway** if a Windows® security message appears.

Installation Steps

Install the Welch Allyn® Diagnostic Cardiology Suite Manager software

1. Plug the USB flash drive into a USB port on your computer.
2. Navigate to the Welch Allyn® Diagnostic Cardiology Suite software package.
3. Double-click on the application file **Manager Full Setup.exe**. The *Manager Setup* dialog window appears prompting installation.
4. Click **Install**.
5. If a Windows® security message appears, click **Yes** to allow the program to make changes to the computer.
6. When the Welch Allyn® Diagnostic Cardiology Suite installation wizard appears, click **Next**.

7. Enter the serial number from the Product ID card that shipped with the product, then click **Next**. Select the desired language from the drop-down, then click **Next**.
8. Select the desired installation option:
 - a. Click **Simple** if you will primarily be printing exam results or saving the exam results as PDFs to a local directory, or
 - b. Click **EMR Integrated** to launch the application directly from a supported Electronic Medical Record (EMR) system.

Note At a minimum, this option requires that you follow detailed instructions presented in the DFU or on the website, or that you contact technical support for assistance.
9. On the next screen, select the desired installation option:
 - a. Click **Full Install** to install both Server and Client systems, or
 - b. Click **Client** to install a Client system.

Note At a minimum, this option requires that you follow detailed instructions presented in the DFU or on the website, or that you contact technical support for assistance.
10. Select the desired PDF Export location:
 - a. Click **Next** to choose the default location ("C:\ConnexCardio"), or
 - b. Click **Browse...** to select a folder location. Then click **OK**.
11. Click **Next** to accept the default Server Port configurations, or contact your network administrator for help.
12. Click **Install** and wait for the installation to complete.
13. When the **Installation Successfully Completed** window appears, click **Restart**.
14. Proceed to the Modality software installation.


Note Welch Allyn® Diagnostic Cardiology Suite supports additional types of deployment such as launch from inside an EMR and setting up a common database. Consult the user manual for further details.


Installation Quick Reference

Install the Modality software

1. Navigate to the Welch Allyn® Diagnostic Cardiology Suite software package.
2. Double-click on the **Windows Installer Package** ("ECG Installation.msi," "Spirometry Installation.msi," etc.).
3. When the setup wizard window appears, click **Next**.
4. Enter the serial number from the Product ID card that shipped with the product, then click **Next**.
5. When the **Ready to Install...** window appears, click **Install**.
6. When the **Completed** window appears, click **Finish**.
7. Repeat these steps for each Modality.

Initial Configuration

1. Log into the Welch Allyn® Diagnostic Cardiology Suite.
2. To set up your preferences, click  (System Configuration) on the main screen. A window appears featuring the pane of buttons shown to the right.
3. See the *Welch Allyn® Diagnostic Cardiology Suite Manager User Manual*, System Settings section, for steps to define the following:
 - a. User permissions
 - b. Modality settings
 - c. File Exchange
 - d. Other administrative functions.

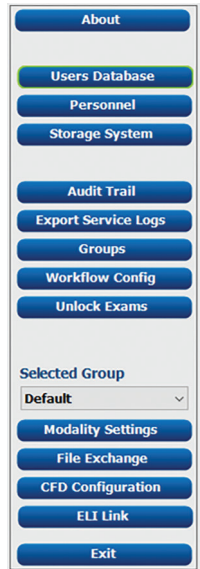

See the *Welch Allyn® Diagnostic Cardiology Suite Manager User Manual*,
REF 80023962, for more detailed Installation and System Configuration Instructions.

Hillrom Technical Support

hillrom.com/en-us/about-us/locations/

Welch Allyn, Inc. is a subsidiary of Hill-Rom Holdings, Inc.

