## ENGLISH

### Guidelines for safe use of the Green Series Portable Headlight

#### Intended use

This headlight is designed to meet the various needs of the physician's office, hospital environment, and specialist's office.

#### Warnings

**WARNING** Green Series Portable Headlight is not suitable for use in the presence of a flammable anesthetic mixture with air or oxygen or nitrous oxide. An explosion may result.

**WARNING** INTENSE LIGHT: Lamp is bright. Do not stare at the lamp when in use. May be harmful to eyes.

#### Cautions

**Caution** Read Operating Manual for Warnings, Cautions, and Instructions for Use.

**Caution** No modification of this equipment is allowed.

**Caution** Do not block access to the wall socket. Ensure that transformer can be unplugged from the AC power source.

**Caution** Use only Green Series Portable Headlight Power Source/Battery Charger and rechargeable battery pack supplied by Welch Allyn.

**Caution** There are no user-serviceable parts inside the device except the rechargeable batteries and headband. Contact the Authorized Service Center listed at [www.welchallyn.com/support](http://www.welchallyn.com/support) for further assistance.

#### Setup and operation

The Green Series Portable Headlight ([#] 46070) consists of the Portable Headlight ([#] 46000) with Direct Power Source ([#] 74170).

The Green Series Portable Headlight is designed to operate off the battery or direct power source. To operate the headlight using the direct power source, follow the instructions and illustrations in the section labeled Direct Plug-In Transformer. Press the on/off switch located on the Direct Plug to operate the headlight.

To adjust the light spot size, turn the knob on the front section of the luminaire. To adjust the rigid headband, simply rotate the adjustment knob located on the back of the headband.

Follow the assembly instructions listed below and the corresponding illustrations.

#### Direct Plug-In Transformer (See Figure 2)

1. Plug Power cord into the port on the top of the luminaire.
2. Push until firmly in place.
3. The instrument is now ready for use.
4. Plug direct power source into a nearby receptacle.

#### Battery Replacement (See Figure 3)

1. Drop the battery pack into the luminaire visor.
2. Plug into the connector.

#### Headband Replacement (See Figure 4)

Holding the luminaire with the on/off switch in the up position, place the visor section of the luminaire into the openings located in the front section of the terry cloth headband.

#### Cleaning (Recommended monthly)

**Caution** Do not immerse in cleaning solutions.

**Caution** Excess solution entering the luminaire assembly could damage internal components. Use caution to ensure cloth is not saturated with solution.

**Caution** Do not sterilize the unit.

1. Unplug the unit prior to cleaning.

2. Wipe down the entire luminaire with a cloth slightly dampened with a mild solution of detergent.
3. Do not operate the light until the light is thoroughly dry.

#### Troubleshooting

Trouble Area	Possible Cause	Corrective Action
No Light Output	Dead battery or battery not connected properly	Verify that the unit operates on direct power source. If yes, insure that the battery is connected properly; refer to the battery replacement section for instructions on connecting the battery. If the battery is connected properly, recharge or replace battery
Low Light Output	Low charge on battery	Verify that the unit operates on direct power source. If yes, recharge or replace battery

#### Specifications

Color Temperature: 5300K

Light Intensity: 40 Lumens

Working Distance: 16 Inches

#### Electrical

Ref Model	Input Voltage	Output Voltage	Output Current
46070(R)	120VAC 50-60Hz	5 volts vdc	1300 mA
46072(R)	230VAC 50-60Hz	5 volts vdc	1300 mA
46074(R)	230VAC 50-60Hz	5 volts vdc	1300 mA
46076(R)	240VAC 50-60Hz	5 volts vdc	1300 mA

#### Environment

	Operating	Storage/Transport
Temperature	32° F - 104° F (0° C - +40° C)	--4° F - 120° F (-20° C - +49° C)
Humidity	75% RH Max continuous	95% RH Max continuous
	500hPa - 1060 Altitude	500hPa - 1060 Altitude

**Equipment Classification:** Class II, Continuous Operation

#### Accessory Parts List

Model 46070 Portable Headlight	Model 74172 Battery Pack with Charger (European)
Model 05232-U Soft Case	Model 74174 Battery Pack with Charger (U. K.)
Model 49642 Soft Terry Cloth Headband, case of 10	Model 74176 Battery Pack with Charger (Australia)
Model 74170 Battery Pack with Charger (Domestic)	Model 72270 Replacement Battery

#### Emissions and Immunity Information

##### EMC compliance

Special precautions concerning electromagnetic compatibility (EMC) must be taken for all medical electrical equipment. This device complies with IEC 60601-1-2:2007.

- All medical electrical equipment must be installed and put into service in accordance with the EMC information provided in this document and *Green Series Portable Headlight System Directions for Use*.

- Portable and mobile RF communications equipment can affect the behavior of medical electrical equipment.

The Green Series Portable Headlight System complies with all applicable and required standards for electromagnetic interference.

- It does not normally affect nearby equipment and devices.
- It is not normally affected by nearby equipment and devices.
- It is not safe to operate the *Green Series Portable Headlight System* in the presence of high-frequency surgical equipment.
- It's good practice to avoid using the headlight in extremely close proximity to other equipment.

For information about electromagnetic compatibility (EMC), see the Welch Allyn website: [http://www.welchallyn.com/apps/products/product\\_category.jsp?catcode=LTG](http://www.welchallyn.com/apps/products/product_category.jsp?catcode=LTG)

#### Product Disposal

Do not dispose of this product as unsorted municipal waste. Prepare this product for reuse or separate collection as specified by local regulations.

#### Warranty

One year.

#### Service Information

For Technical Support or to obtain information about any Welch Allyn product, contact Welch Allyn Technical Support:

([www.welchallyn.com/support](http://www.welchallyn.com/support)) or visit [www.welchallyn.com/about/company/locations.htm](http://www.welchallyn.com/about/company/locations.htm)

For patent information, please visit [www.welchallyn.com/patents](http://www.welchallyn.com/patents).

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# Welch Allyn®

## Green Series Portable Headlight System

[#] 46070, 46072, 46074, 46076, 46070R, 46072R, 46074R, 46076R

### Directions for use

This manual applies to: [REF] 901069 Headlight

### Welch Allyn Technical Support Service Information

[www.welchallyn.com/about/company/locations.htm](http://www.welchallyn.com/about/company/locations.htm)

Welch Allyn, Inc.  
4341 State Street Road  
Skaneateles Falls, NY 13153 USA  
[www.welchallyn.com](http://www.welchallyn.com)

Mat. No. 728481, 80023356 Rev. A  
Revision date: 2018-07

© 2018 Welch Allyn. All rights are reserved. To support the intended use of the product described in this publication, the purchaser of the product is permitted to copy this publication, for internal distribution only, from the media provided by Welch Allyn. No other use, reproduction, or distribution of this publication, or any part of it, is permitted without written permission from Welch Allyn. Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product, that may result from failure to use this product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual.

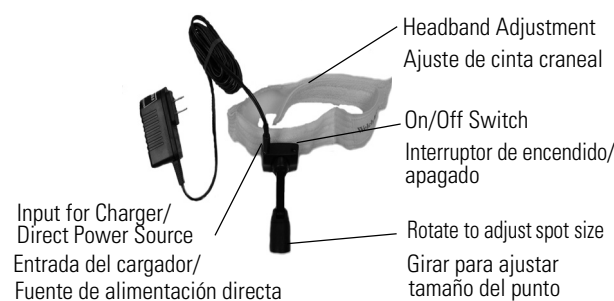
Figure 1/Figura 1

Figure 2/Figura 2

Figure 3/Figura 3

Figure 4/Figura 4

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